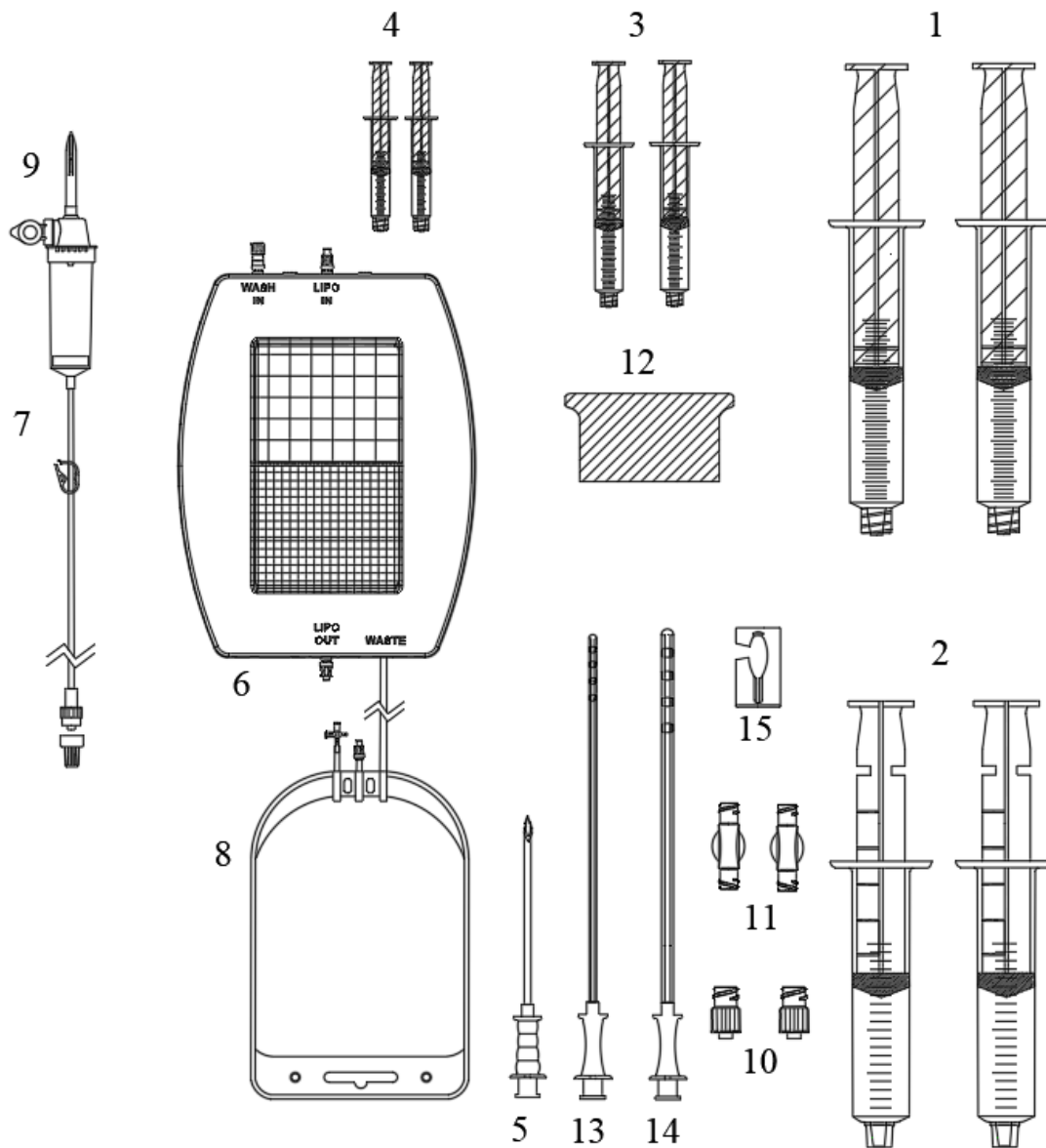
LIPO-PACK DUO

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
EN

LIPO-PACK DUO

System for aspiration, processing and grafting of autologous adipose tissue

CE 0123

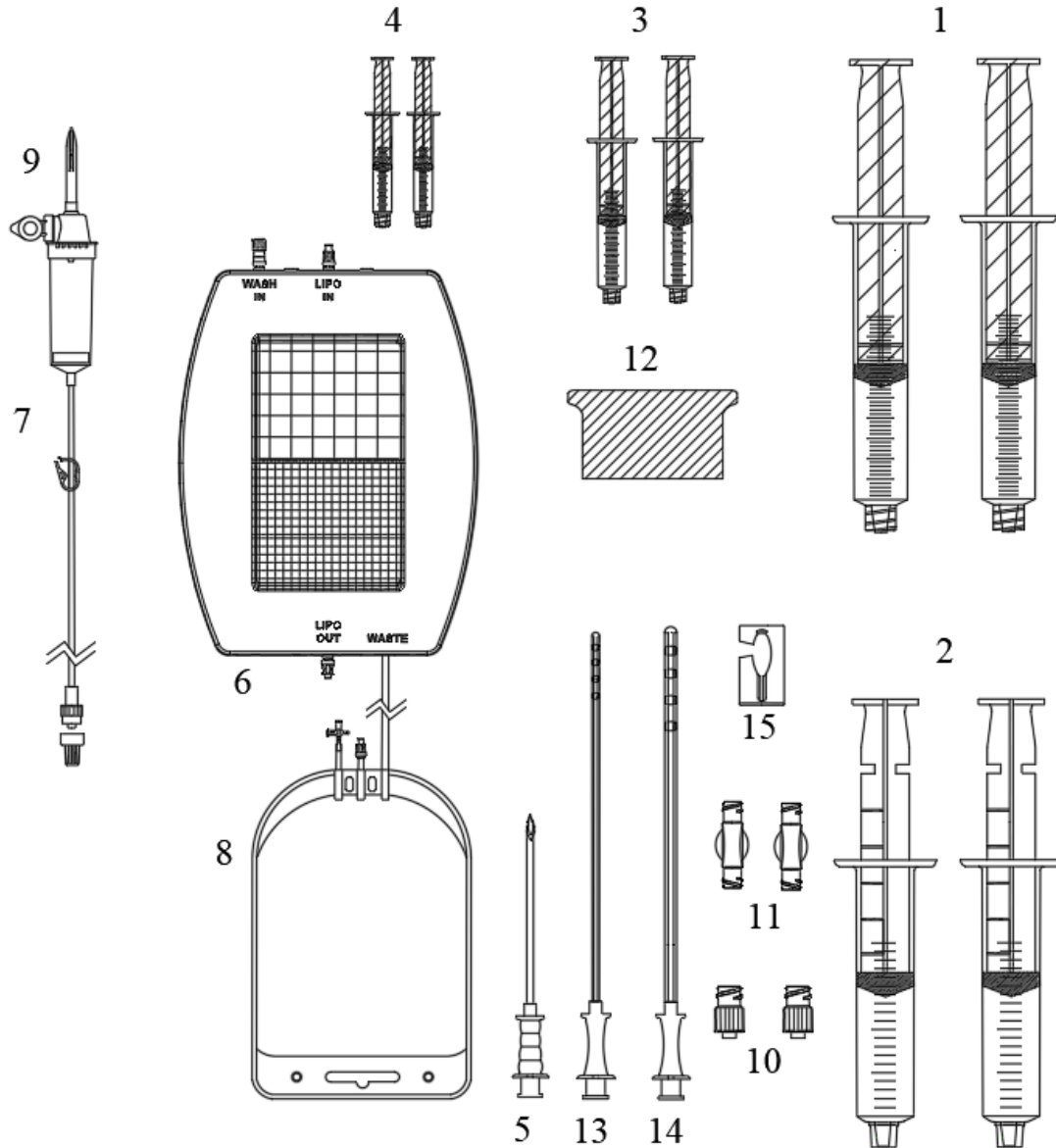KIT CONTENT:

1. 2x 60ml syringes for the injection of Klein solution.
2. 2x 60ml syringes for liposuction (recognizable by the presence of the block on the plunger).
3. 2x 10ml syringes for the recovery of 'treated' fat.
4. 2x 3ml syringes for the injection of 'treated' fat.
5. 1x replanting needle to be connected to the injection syringes.
6. 1x filter bag for lipo-aspirated wash.
7. 1x Infusion set with tip to puncture saline bag with clamp.
8. 1x Waste collection bag already assembled with the filter.
9. 1x Outflow punch.
10. 2x Combi caps closed LLM-LLF
11. 2X double female Luer fitting LLF-LLF
12. 1 spatula.
13. 1x Metal cannula for liposuction to be used in combination with a syringe – diam.2.5mm (13G).
14. 1x Metal cannula for the injection of Klein solution to be used with a syringe – diam.1.6mm (16G).
15. 1x Open slide clamp.

Instructions for use

The various operating steps to be performed are as follows:

IMPORTANT: Before starting the procedure close the outflow tap of the waste bag.

a) Local anaesthetic injection

Fill the syringes (1) with Klein solution. Connect the 60ml anaesthetic injection syringes (1) to the 1.6mm Ø metal injection cannula (14).

Insert the thus assembled syringe through an access 15cm to the side of the belly. The direction of the cannula must be parallel to the skin and it must be kept in the subcutaneous abdominal fat layer. Completely insert the metal cannula and make forward, backward and fanlike movements in such a way as to cover the entire drawing area.

Repeat this operation 2-3 times.

Leave the Klein solution to act for at least 7-8 minutes or for the time indicated by the manufacturer.

b) LIPO-PACK DUO preparation

Drop the waste collection bag (8) "WASTE" to the ground.

Perforate the bag of saline solution with the tip (9), press the soft part a couple of times and then open the blue cap located on one side. The tip (9) must be connected through the tube to the "WASH IN" valve of the bag (6).

Fill the filter bag with saline solution for a few minutes and then stop the flow using the hose clamp located between the tip (9) and the bag (6).

c) Liposuction

Connect the liposuction syringe (2) to the metal cannula (13). Insert the cannula (13) and pull the plunger of the syringes (2) as far as it will go. Maintaining the suction, move the cannula forward and backwards in such a way as to cover a large area and draw a good amount of adipose tissue. Repeat the operation until the two syringes (2) are filled.

Suture the abdominal access if necessary.

d) Filtration

Connect the lipoaspiration syringe (2) to the LIPO IN input of the bag (6) and transfer the entire content into the filter. Open the inlet tube by releasing the hose clamp. Filter the aspirated material with the aid of the spatula (12).

Close the inlet tube by locking the hose clamp. Connect a purified fat injection syringe (5 or 6, depending on the desired volume) to the outlet (LIPO OUT) and fill it with the filtered material with the aid of the spatula (12).

Repeat the entire filtration operation (d) for each liposuction syringe available.

e) Cell grafting

Note: If the consistency of the aspirated material is too dense for easy injection into the site to be treated, you can join two syringes (3) with an FL/FL connection and then further fragment the aspirated material with alternate movements. Otherwise, the material can be used directly. Transfer the material from the syringe (3) to the syringe (4) via the LF / LF connection, if necessary use the combi caps (10) to close the syringes.

Connect the tip (5) to the syringes (4) and inject the filtered material into the site to be treated.

Indications

The LIPO-PACK DUO device is intended for ASPIRATION, PROCESSING AND GRAFTING OF AUTOLOGOUS ADIPOSE TISSUE To be used under medical supervision only.

The user doctor must verify for each type of application the homo-functionality of use of the adipose tissue obtained by filtration.

Precautions for use

- The device may exclusively be used by physicians.
- After use, the needle may constitute a risk of contamination and/or infection. Therefore, handle with care and dispose of the product in accordance with the regulations in force and medical practice.
- Through the use of the device you get an autologous biological product and as such we do not recommend treatment in patients with malignancies in progress and state of pregnancy.
- The device is to be considered for autologous and single-use use. Re-processing of the medical device is not permitted.
- In the first days after surgery you may note a slight swelling in the adipose withdrawal zone. The specialist may prescribe, according to each clinical case, a brief rehab programme. We recommend a compressive medication in the withdrawal zone for at least a week.

PACKAGING

The device is packed in a medical-grade paper and transparent laminated film bag contained in a box. This bag contains all the kit components in their turn individually packed in medical-grade paper and transparent laminated film bags. The entire system contained in the outer bag is sterilized by ethylene oxide (EtO) in compliance with ISO 11135 requirements. It is sold in packs of 1 piece.

The device is not reusable. If it is reused, there is a risk of altering device performance and/or the possibility of cross-contamination. Do not use the device if the package is open or damaged or contaminated.

Biopsybell declines all responsibility for any damage caused by improper use or different from what is indicated in this instruction leaflet.

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